

Accu-Chek Multiclix.



Easy steps for gentle testing



1

PRESS

Press down like a ballpoint pen until the button turns yellow.



2

CLICK

Click yellow button.

Some models have white display bars.

Need a new lancet?



TWIST

Do the double twist. Numbers show how many lancets are left.

• Accu-Chek Multiclix is quick and easy to use, with no lancets to see or handle

Changing the depth setting and lancet drum with Accu-Chek Multiclix.

Choose your depth setting



SET PENETRATION DEPTH

Select depth by turning dial.
For beginners, select a lower setting (start with level 1).



You can perform six tests before you have to change the lancet drum



LOAD DRUM

Remove cap, insert new drum with blue ring end first (contains 6 lancets). Snap drum into place. Replace cap.



CHANGE DRUM

When the number 1 appears in display, the last lancet is in use.* Remove cap. Remove used drum (has red stripe). *Used drums cannot be re-inserted.* Load new drum.

*In models with white display bars, no bars will show in display.



accu-chek.ca

ACCU-CHEK and ACCU-CHEK MULTICLIX are trademarks of Roche.
© 2010 Roche Diagnostics. All rights reserved.

Accu-Chek Multiclix lancing device is intended for use by one person only.

ACCU-CHEK®